EN	ACUVUE™ RevitaLens Multi-Purpose Disinfecting Solution Exceptional Disinfection and All-Day Comfort	2
DE	ACUVUE™ RevitaLens All-in-One-Desinfektionslösung Herausragende Desinfektionsleistung und ganztägiger Komfort	3
FR	Solution de décontamination multifonctions ACUVUE™ RevitaLens Grande efficacité décontaminante et confort toute la journée	4
IT	Soluzione disinfettante multiuso ACUVUE™ RevitaLens Disinfezione ottimale e comfort per tutto il giorno	5
NL	ACUVUE™ RevitaLens desinfecterende all-in-one contactlensvloeistof Uitstekende desinfectie en de hele dag comfort	6

AMO Ireland, Block B, Liffey Valley Office Campus, Quarryvale, Dublin, Ireland
CH Rep: AMO Switzerland GmbH, Gubelstrasse 34, 6300 Zug, Switzerland
TETRONIC is a trademark of BASF Corporation. Patents: www.jnjvisionpro.com/patents
© Johnson & Johnson Vision Care Companies 2025

EN ACUVUE™ RevitaLens Multi-Purpose Disinfecting Solution

Exceptional Disinfection and All-Day Comfort

Important – Please read carefully and keep this package insert for future reference.

INTENDED PURPOSE: Clean, rinse, disinfect, store, lubricate, remove surface deposits including lipids and proteins, and re-wet soft (hydrophilic) contact lenses.

INTENDED USERS: ACUVUE™ RevitaLens Multi-Purpose Disinfecting Solution is intended to be used by wearers of soft (hydrophilic) contact lenses (also with lens case).

INDICATIONS: ACUVUE™ RevitaLens Multi-Purpose Disinfecting Solution cleans, rinses, disinfects, stores, removes surface deposits including lipids and proteins, and re-wets soft (hydrophilic) contact lenses.

CONTRAINDICATIONS: None known.

CLINICAL BENEFITS: Cleaning and disinfection of contact lenses. Provides all-day comfort contact lens wear.

Pregnant or nursing women were not included in the clinical trial(s) conducted.

DIRECTIONS FOR CONTACT LENS DISINFECTION:

1. Wash hands with soap & warm water and dry with a lint-free towel before handling lenses and/or lens case, to reduce the chance of getting an infection.
2. After removing your contact lenses, clean and disinfect your lenses by placing 3 or more drops of solution on one side of lens surface.
3. RUB lens gently in a non-circular motion from the center to the edges for 2 to 4 seconds.
4. Turn the lens over and repeat steps 2 & 3.
5. RINSE each side of lens for 5 seconds with fresh solution. Place lenses in clean lens case and fill with solution. Close case tightly.
6. SOAK lenses for a minimum of 6 hours for disinfection, cleaning, and protein removal.

Disinfecting Solution:

- Place lenses in the correct chambers of the lens case (blue for left - white for right).
- Fill both chambers with fresh ACUVUE™ RevitaLens Multi-Purpose Disinfecting Solution. Replace lens case caps and soak for at least 6 hours for disinfection, protein and lipid removal. Your lenses are now ready for wear. Rinse the lens case with ACUVUE™ RevitaLens Multi-Purpose Disinfecting Solution and leave to air dry upside down on a clean towel.

Attention: You may leave your lenses in the unopened lens case in ACUVUE™ RevitaLens Multi-Purpose Disinfecting Solution for up to 30 days continuously. After 30 days, cleaning and disinfection must be repeated before wearing your lenses.

PROTEIN REMOVAL FROM YOUR CONTACT LENSES

ACUVUE™ RevitaLens Multi-Purpose Disinfecting Solution removes protein while your lenses are soaking in the lens case. Daily use of ACUVUE™ RevitaLens Multi-Purpose Disinfecting Solution eliminates the need for a separate enzymatic cleaner for many lens wearers - check with your eye care professional.

WARNINGS:

All instructions must be followed. Failure to do so may result in serious eye infection or injury. If eye irritation occurs immediately discontinue lens wear and contact your eye care professional.

Do not put ACUVUE™ RevitaLens Multi-Purpose Disinfecting Solution directly from the bottle into your eye.

PRECAUTIONS:

General: Use ACUVUE™ RevitaLens Multi-Purpose Disinfecting Solution with soft contact lenses, including silicone hydrogel lenses. Do not use with hard (rigid gas permeable, RGP) lenses. Do not change your contact lens care system without consulting your eye care professional. Do not use if allergic to any of the ingredients. Borates (boric acid & sodium borate) are defined as CMR 1B substances in a concentration above 0.1% weight by weight and are safe when the product is used according to label instructions. Problems with contact lenses and lens care products could result in corneal infection and/or ulcers and lead to loss of vision. It is essential that you follow your eye care professional's directions and all labeling instructions for proper use of lenses and lens care products, including the lens case. Do not wear your lenses longer than prescribed. Discard any lenses that are damaged or dehydrated. Avoid using aerosols when wearing contact lenses. Attend regular check-ups with your eye care professional. Keep out of the reach of children.

Use: Always use ACUVUE™ RevitaLens Multi-Purpose Disinfecting Solution with the non-sterile lens case provided in the packaging.

Do not use if tamper-evident seal is missing or broken.
Do not reuse solution; always use fresh solution.
Close the bottle tightly immediately after use.
Always manually rub your lenses with solution for at least 2-4 seconds before soaking.
Do not swim, shower or bathe while wearing contact lenses.
Do not use saliva or put lenses in your mouth to rewet.
To avoid contamination do not allow the bottle tip to touch any surface and avoid contact with eyes.
Do not swallow the solution. For accidental ingestion seek medical help, if necessary.
Do not rinse your lenses/lens case and/or store lenses with water or any non-sterile solution.

Storage: Store between 5°C to 25°C.

Discard

and Expiry: Use before the expiry date on label and carton. Do not use after the expiry date. Discard any remaining solution 90 days after first opening the bottle. Discard any remaining solution in your contact lens case before disinfecting your lenses again. Always discard your old lens case and use the new lens case provided. Dispose all products used as per the local requirement(s).

Ingredients: Alexidine 0.00016%, Polyquaternium-1 0.0003%, Boric acid, Sodium borate decahydrate, TETRONIC 904, Edetate disodium (EDTA), Trisodium citrate dihydrate, Sodium chloride and Purified water.

REPORTING OF ADVERSE REACTIONS (SIDE EFFECTS):

Any incident experienced whilst wearing this product should be reported to the manufacturer and/or its authorized representative and/or to your national authority.

Symbols

	Consult instructions for use		Manufacturer		Date of manufacture		Batch code		Use by date
	Sterilized using aseptic processing techniques		Single sterile barrier system		Medical device		Catalogue number		Temperature limit
	Caution		Contains hazardous substances		Do not use if package is damaged and consult instructions for use		Discard any remaining solution 90 days after first opening the bottle		

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